CONCARE® CLASSIC Reiniger

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EN: Instructions for use

CONCARE® CLASSIC cleaner is used for the daily cleaning of rigid and gas-permeable contact lenses.

The cleaning effect of CONCARE® CLASSIC cleaner is guaranteed by small particles present in the solution, which eliminate deposits from the contact lens.

INSTRUCTIONS FOR USE:

1. Wash your hands.
2. Shake the bottle before each use.
3. Place the lens on the palm of your hand and apply a few drops of CONCARE® CLASSIC cleaner to the lens.
4. Rub it carefully for about 10-20 seconds between your fingertips.
5. Rinse the lens thoroughly with CONCARE® CLASSIC conditioner or CONCARE® saline solution to remove the CONCARE® CLASSIC cleaner completely.
6. Fill the lens case with CONCARE® CLASSIC conditioner and leave the lens there overnight or for at least 6 hours.

PRECAUTIONS:

- The product is not suitable for soft contact lenses.
- This solution should not come into contact with the eyes. If this occurs, wash them with plenty water.
- Close the bottle tightly after each use.
- Do not touch the dropper with your fingers.
- Check that the seal is intact before using the product for the first time.
- Use the product within 90 days after opening.
- Keep out of the reach and sight of children.
- Do not use this product if you are allergic to any of its ingredients.

INGREDIENTS:

Sodium chloride, lauryl ether sulphate, silica gel, titanium dioxide, boric acid, sodium tetraborate, 0.1 % disodium edetate and 0.001 % polyhexamethylene biguanide in purified water

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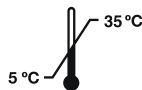
Cleaning solution

For rigid and permeable contact lenses

Disop
Avda. Veleparrá, 31-A
28108 Madrid, España

CE 0318

Vertrieb/ Distributeur
Distributore/ Distributor
Distributie/ Distribuidor:
Hecht Contactlinsen GmbH
Dorfstraße 2-4, D-79280 Au

STERILE A